


STEP 1: iCare EIDON® Ultra Widefield Camera Setup

- + **Turn on iCare EIDON:** Power button on backside of device
- + **Log In ID** = Doctor
- + **Password** = doc
- + **Click EIDON:** Yellow icon

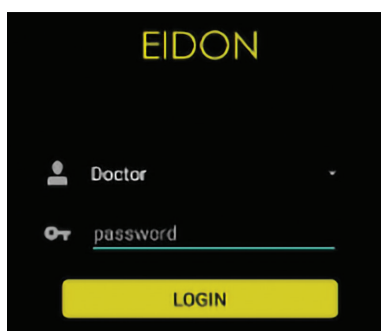


STEP 2: Set Up Patient

IF **New** Patient

- + **Click New Patient:** Upper right corner
- + **Enter:** First/Last Name, Patient ID, DOB
- + **Click:** Save

IF **Existing** Patient



- + **Search Name:** Use magnifier icon

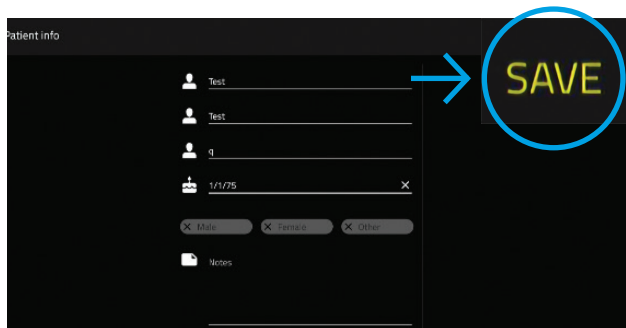

STEP 3: Image Capture

- + **Click:** New Exam
- + **Click Single Field:** Change to **Multiple fields**
 - **Click:** Central Nasal – 1, then
 - **Click:** Temporal – 2
 - **Click:** Central field to deselect
- + **Click:** Start

USEFUL TIP: iCare EIDON is **fully-automatic**, the camera automatically aligns, focuses, and captures

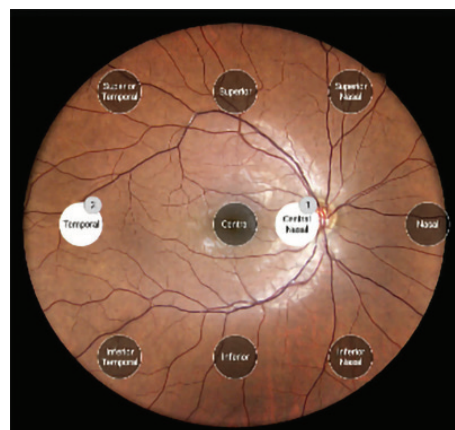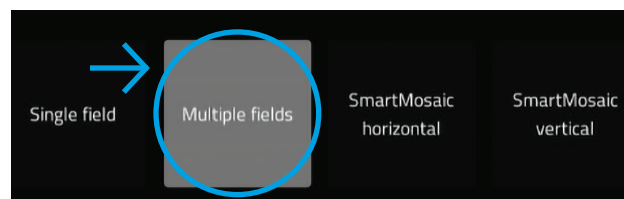


To obtain service or information please call Customer Service at (888) 422-7313 Option 3, then Option 2 for Imaging Service.

STEP 4: Capture Workflow



- + **Before clicking proceed**, ensure the patient's chin is comfortably on the chinrest and their forehead is against the forehead rest.
- + Ask the patient to **blink normally**, open eyes wide between blinks and follow the green light. There will be two flashes on the right eye before the instrument automatically moves to the left eye.
 - Capture starts automatically
 - Camera aligns, focuses, and captures retinal image

USEFUL TIP: During image capture, iCare EIDON will beep a few seconds before taking the picture. When you hear the beep, remind the patient to **“open eyes wide.”** This will help ensure a clear, high-quality image is obtained.

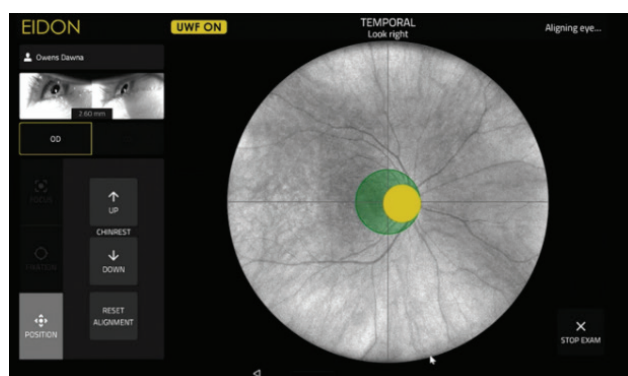


STEP 6: Create Mosaic OU

- + **Click: Each image** so there are yellow checks
- + **Click: Create mosaic**
- + **Once created:** Click **back arrow** on bottom of screen; repeat steps above, click **new mosaic**, then **OS**
- + **Data is saved:** And ready to view in iCare EIDON Remote Viewer

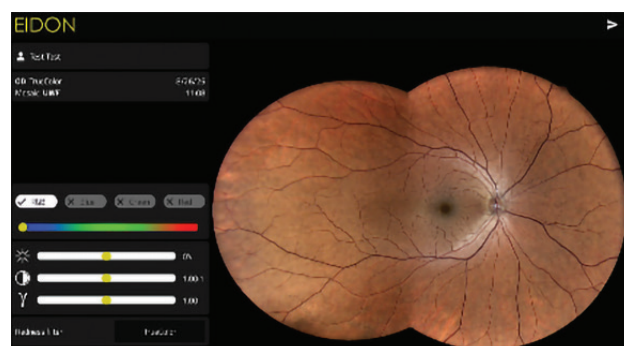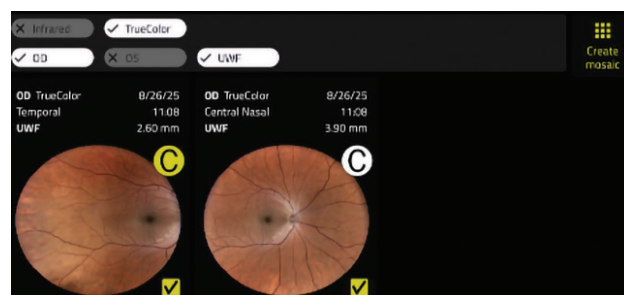

STEP 5: Review & Save

- + **Image save:** Data is automatically saved and ready to view in **iCare EIDON remote viewer**
- + **Image delete:** Images can be deleted post capture by selecting the Image then clicking the trash can

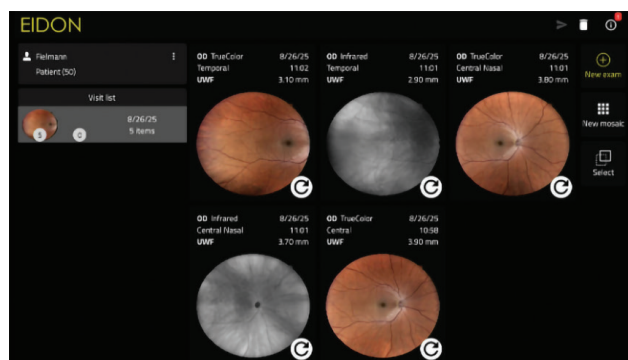

The purpose of this document is to provide tips on the use of iCare EIDON. It should not in any case be substituted for the user manual for operation and is intended only as a quick guide for reference.

For further information, please read the instruction manual. Contact your iCare Sales Representative for support or questions specific to operate your device.